

# Getting Started with the ANRISK® App



| AI-Based Brain Aneurysm Risk Assessment

## Table of Contents

1. Download the app	02
2. Login	02
3. How to register your voucher	03
4. Entering your data	04
5. Your analysis report	06


# 1. Download the app

1. Download ANRISK® from the App Store (iPhone) or the Google Play Store (Android).

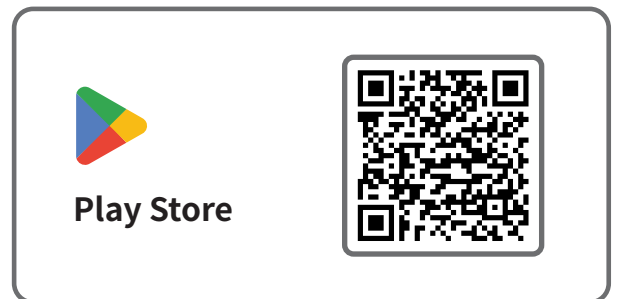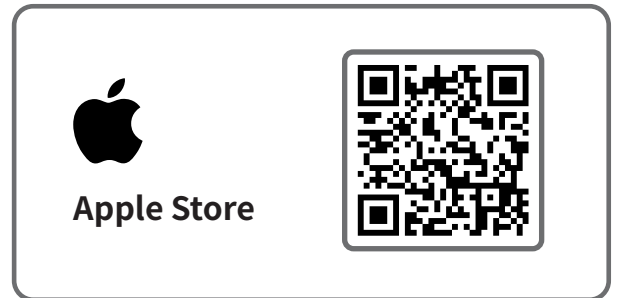


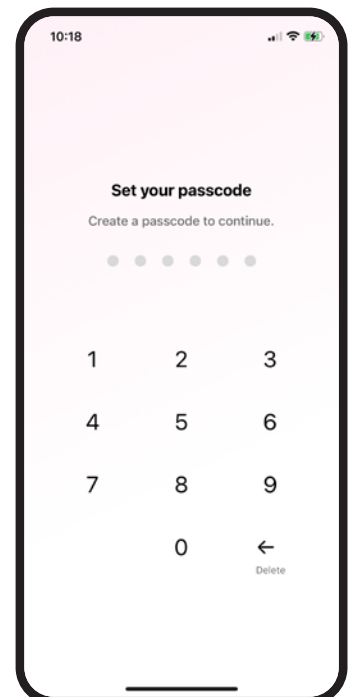
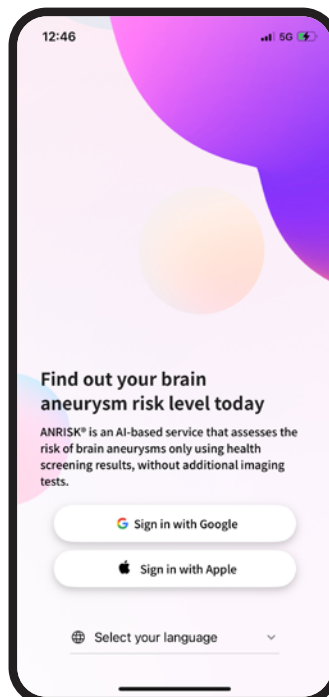
# 2. Login

1. Sign in with your Google or Apple account.

 Please sign up first if you don't have an account.



2. Set or enter your passcode to unlock the app.

 If this is your first time using the app, you'll be prompted to create a passcode before continuing.






### 3. How to register your voucher

1. From the home screen, tap the button in the top right corner, or go directly to the **Settings**.
2. In **Settings**, tap the arrow ‘>’ next to **Analyses**.

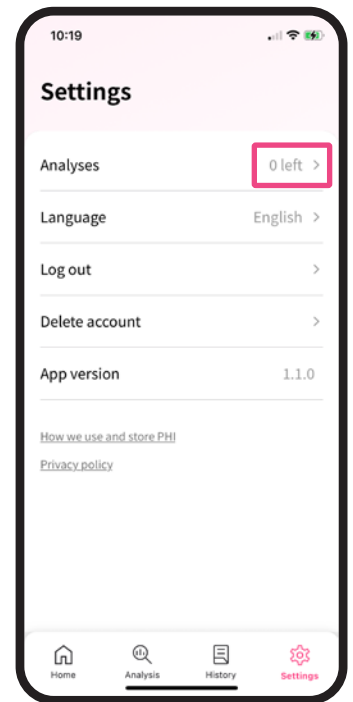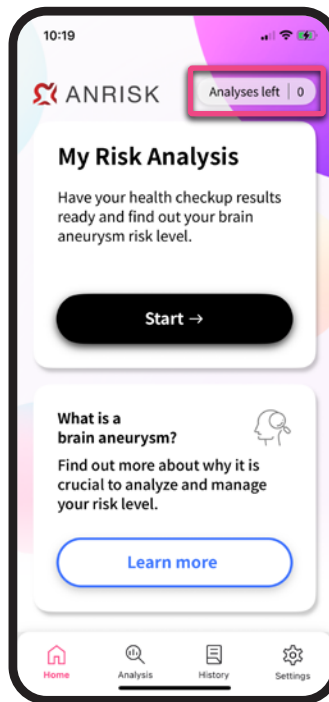



3. In the pop-up, tap **Register voucher** to go to the registration page.
4. Enter the 10-digit number from your voucher and tap **Next** to complete the registration.

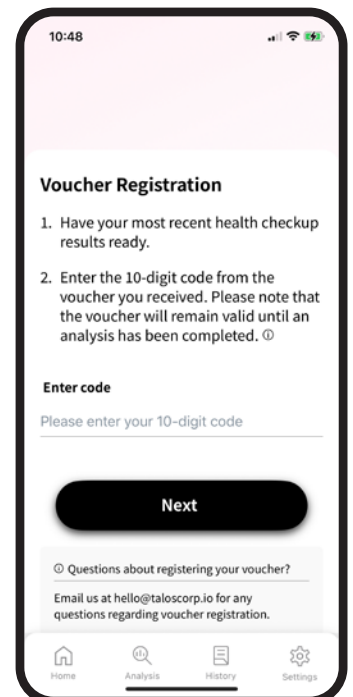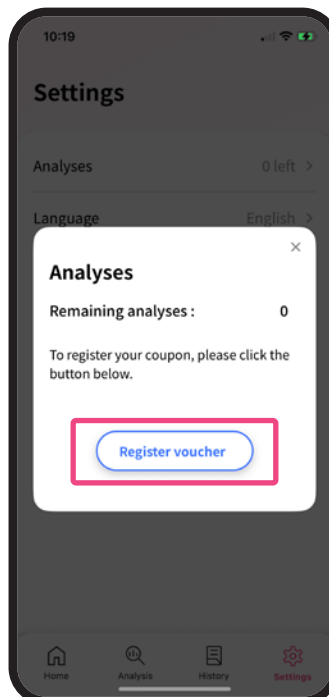


 If you need help or run into any issues, please email us at [hello@talosc corp.io](mailto:hello@talosc corp.io).


## 4. Entering your data

1. From the Home screen, tap **Start** under **My Risk Analysis**.
2. Tap **Next** to input your health check-up data.

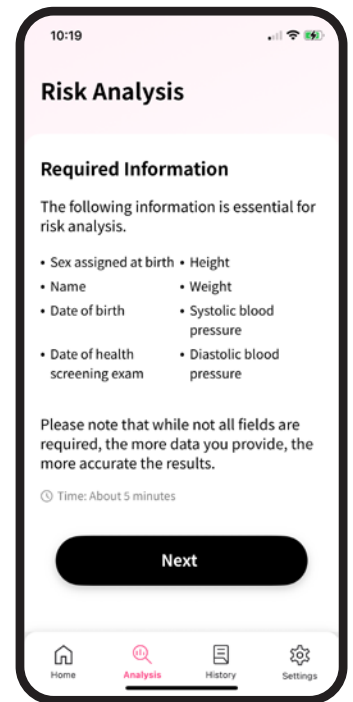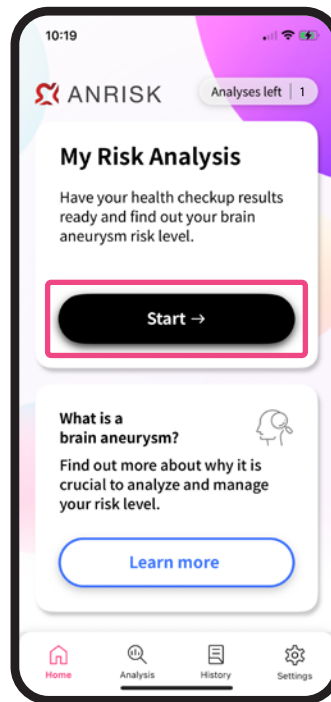

**i** If you have not yet registered a voucher, you'll be redirected to the **Register voucher** page. Enter your 10-digit voucher number to continue.

 If you need help or run into any issues, please email us at **hello@taloscorp.io**.

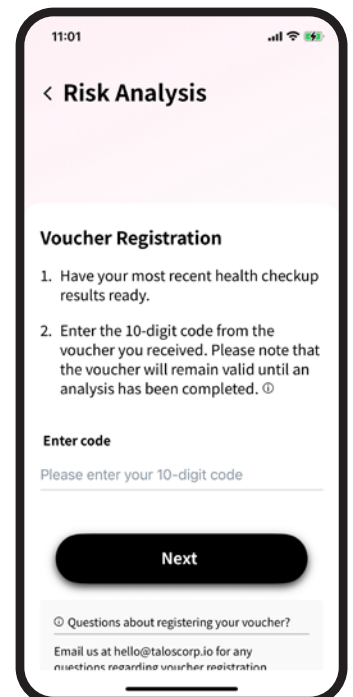

3. Enter your basic information and check-up data.

This screenshot shows the 'Risk Analysis' screen at 10:19. It features a progress bar at the top. Below it are four input fields: 'Name\*', 'Sex assigned at birth\*' (with a dropdown arrow and a note 'Please select an option'), 'Date of health screening exam\*' (with a note '(After 2000-01-01)' and a calendar icon), and 'Date of birth\*' (with a calendar icon). At the bottom are 'Back' and 'Next' buttons. The bottom navigation bar includes icons for Home, Analysis (active), History, and Settings.

This screenshot shows the 'Risk Analysis' screen at 10:35. It features a progress bar at the top. Below it are four input fields: 'Systolic blood pressure\*' mmHg (70 ~ 300), 'Diastolic blood pressure\*' mmHg (36 ~ 200), 'Fasting glucose mg/dL (26 ~ 999)' (with a note 'Leave blank if unknown'), and 'Total cholesterol mg/dL (26 ~ 999)' (with a note 'Leave blank if unknown'). At the bottom are 'Back' and 'Next' buttons. The bottom navigation bar includes icons for Home, Analysis (active), History, and Settings.

4. Answer the short survey on family history and smoking status and scroll down to agree to the **Consent Form** as the final step before submitting your analysis request.

This screenshot shows the 'Risk Analysis' screen at 10:36. It features a progress bar at the top. Below it is a survey question: 'Do you have a family history of stroke?'. There are three radio button options: 'I do not know', 'No', and 'Yes'. At the bottom are 'Back' and 'Next' buttons. The bottom navigation bar includes icons for Home, Analysis (active), History, and Settings.

This screenshot shows the 'Consent Form' screen at 10:36. It contains the following text: 'The purpose of collecting your PHI (Protected Health Information) is for the risk analysis of a cerebral aneurysm. Please note that your data is encrypted before being stored and may be used for research purposes for the improvement of our algorithms and services.' Below this is a paragraph: 'By checking the box below, you are giving consent for the ANRISK® application to collect data and use it for analysis. If you do not wish to proceed, you may withdraw consent and no longer continue with the risk analysis. You may also view and or delete previous assessments from the 'History' page by contacting us at hello@taloscorp.io.' At the bottom is another paragraph: 'To view further information about the data collection, its use, and security, please click on the information icon in the top right corner of the 'Home' page on this application.'

## 5. Your Analysis Report

1. Once the analysis is complete, you'll be taken automatically to the **Analysis History** page. Tap on the analysis date to view details.
2. On the results page, tap on the **Share** '🔗' button in the top-right corner to export your report as a PDF. You can either generate a QR code (save the code by taking a screenshot) or enter a recipient address and share via email.



Please note and links shared via QR code or email are valid for only **1 day**, so be sure to save your report before it expires.

