

Getting Started with the ANRISK® App


| AI-Based Brain Aneurysm Risk Assessment

Table of Contents

1. Download the app	02
2. Login	02
3. Before your begin	03
4. Entering your data	05
5. Your analysis report	07
6. Manage analyses	08

1. Download the app

1. Download ANRISK® from the App Store (iPhone) or the Google Play Store (Android).

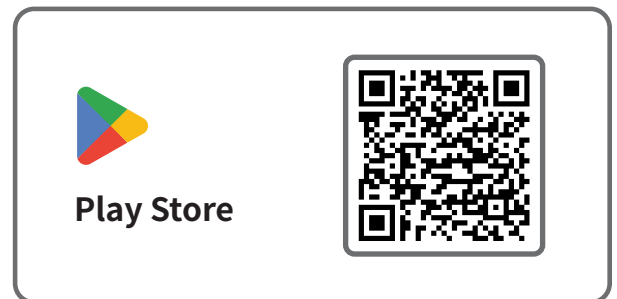



2. Login

1. Sign in with your Google or Apple account.

 Please sign up first if you don't have an account.



2. Set or enter your passcode to unlock the app.

 If this is your first time using the app, you'll be prompted to create a passcode before continuing.

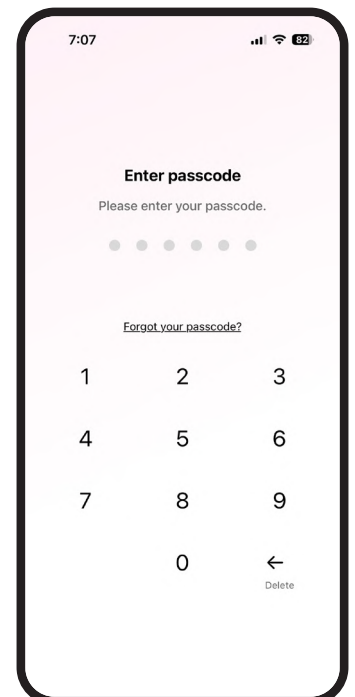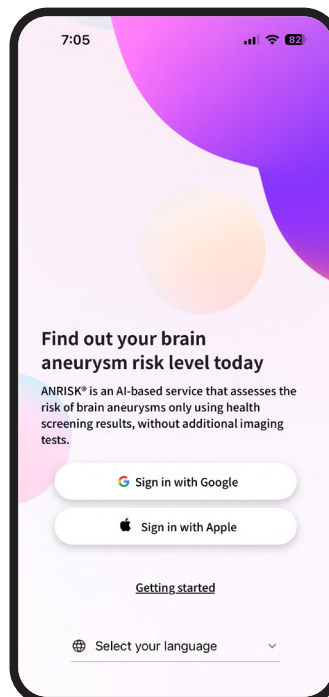

3. Before you begin

Before starting an analysis, you must either register a voucher or purchase analysis credits. Tap the 'Analyses left' button in the top-right corner to view the available options.

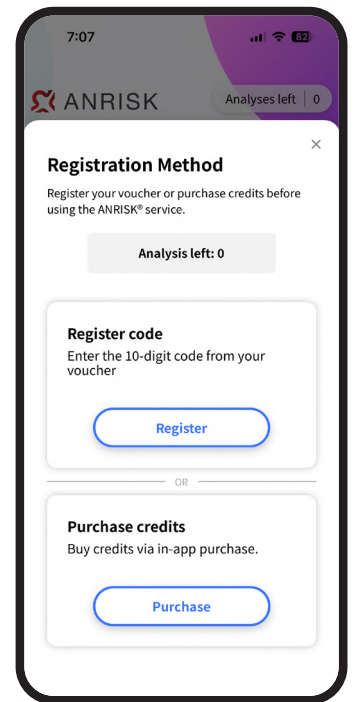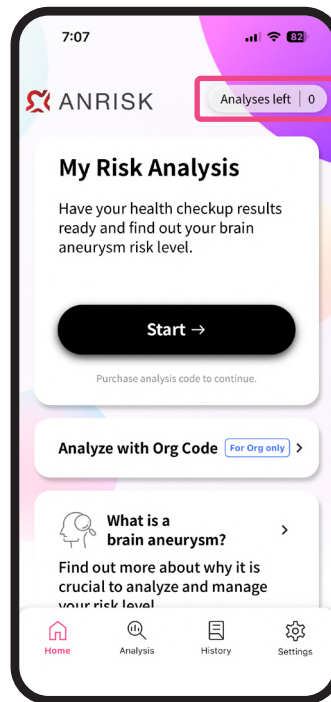



- **Voucher Registration**

Enter the 10-digit code printed on your voucher and tap '+ Register.' Each voucher can be used for one analysis.


- **Analysis Credit Purchase**

Purchase a single credit or a credit package. Each credit allows one analysis and does not expire.

 If you need help or run into any issues, please email us at hello@taloscorp.io.

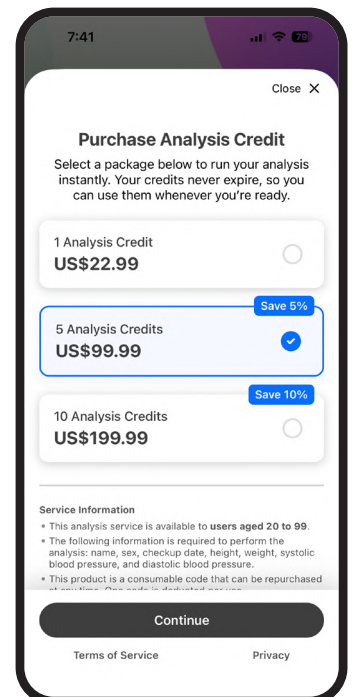
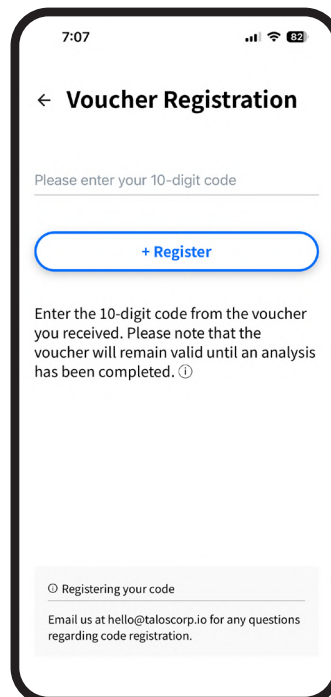


Start your analysis once you have registered your voucher or purchased analysis credits.

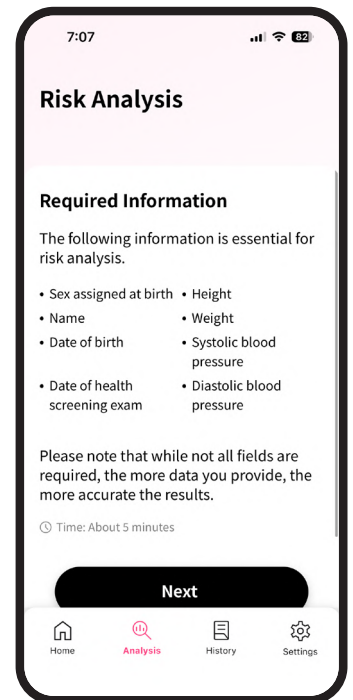





• Organizational Code

1. If you have received an organizational code, tap on 'Analyze with Org Code' from the Home screen to register.
2. Once you enter your 10-digit organizational code, you will be automatically directed to the analysis page.

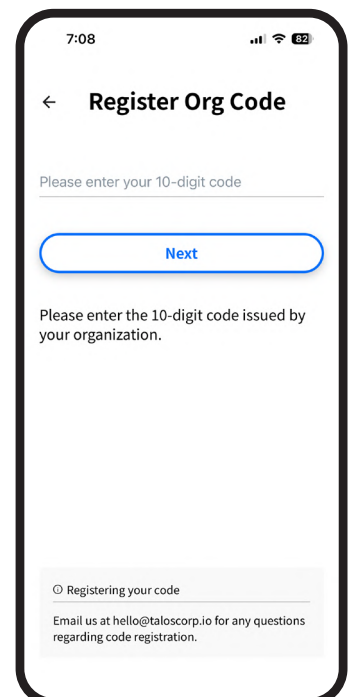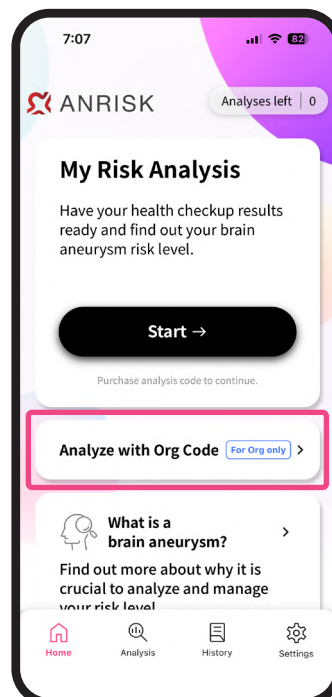

4. Entering your data

1. From the Home screen, tap 'Start' under My Risk Analysis.
2. Tap 'Next' to input your health check-up data.
3. Enter your personal information, as well as your health check-up data.

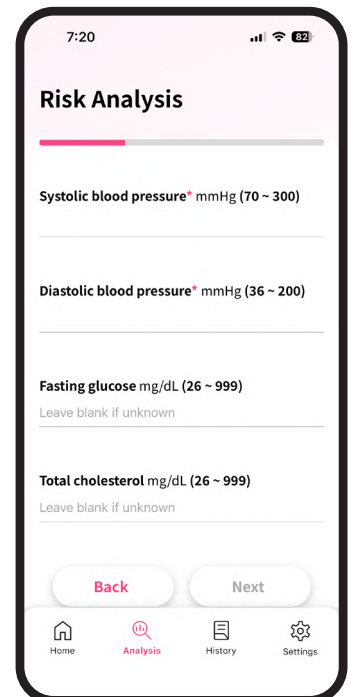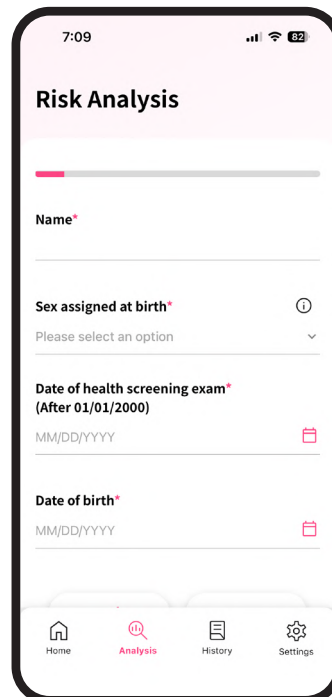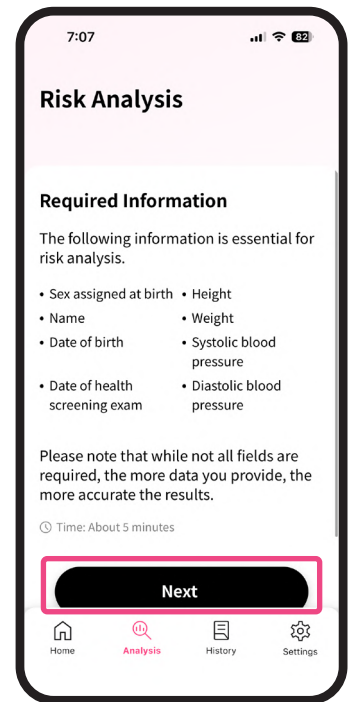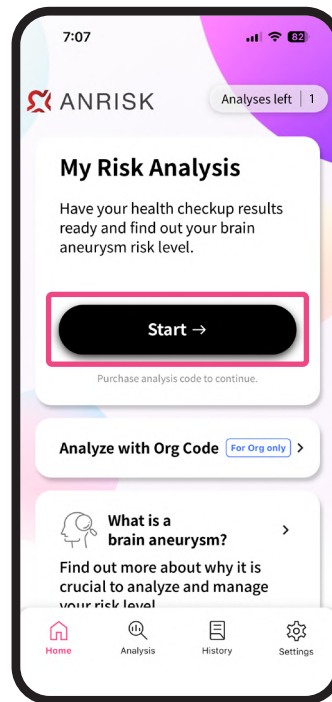

4. In addition to your check-up data, please answer the questionnaire on family history and smoking status. In the final step, agree to the consent for use of one analysis code to submit your analysis request.

The screenshot shows the 'Risk Analysis' screen of the ANRISK app. At the top, the title 'Risk Analysis' is displayed. Below it is a progress bar. The main question is 'Do you have a family history of stroke?'. There are three buttons: 'I do not know' (black), 'No' (white), and 'Yes' (white). At the bottom, there are 'Back' and 'Next' buttons. The bottom navigation bar shows 'Home', 'Analysis' (selected), 'History', and 'Settings'.



The screenshot shows the 'Risk Analysis' screen of the ANRISK app, displaying a consent form. The title 'Risk Analysis' is at the top. Below it is a modal window titled 'Agree to request analysis'. The modal contains two bullet points: 'I agree to use 1 analysis code to request the analysis.' and 'I confirm that all entered information is correct.' There is a checkbox with a checkmark and the text 'I have read and understood the implications of the consent form.' Below this is the date 'Date: Feb 10, 2026' and a blue 'Request analysis' button. The bottom navigation bar shows 'Home', 'Analysis' (selected), 'History', and 'Settings'.

5. Your analysis report

1. Once the analysis is complete, you'll be taken automatically to the **Analysis History** page. Tap on the analysis date to view details.
2. Use the QR code or email icon in the top-right corner of the analysis results page to share the results as a PDF file.



Please note that links shared via QR code or email are valid for only **1 day**, so be sure to save your report before it expires.

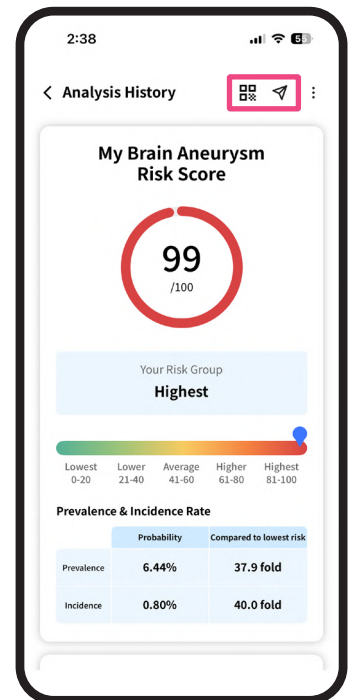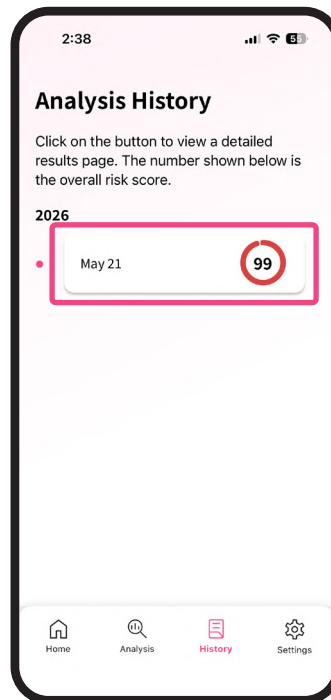

6. Manage analyses

1. View your analysis credit usage, registrations, and purchases on the 'Manage Analyses' page. Access it from 'Settings' in the bottom menu.
2. You may also register your voucher or purchase analysis credits by tapping on the button 'Add analyses'.

